




## MANITOBA RENAL PROGRAM


<b>SECTION: 30.20 Hemodialysis</b> 30.20.05a Appendix A Statlock Application Procedure	<b>CODE 30.20.05a</b>
<b>EFFECTIVE DATE:</b> October 2018	<b>REVISION DATE</b>

### Application Technique

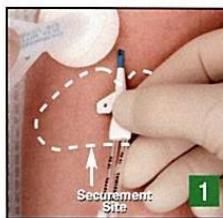


#### Prep

1. Cleanse around catheter insertion and outlined securement site with Chloraprep®, alcohol or as recommended by your health care provider. This is important to remove any lotions or oils from the skin so the anchor pad will hold well.

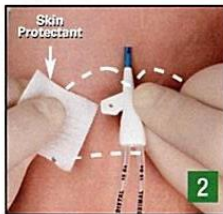


2. Apply skin protectant provided to same securement site, apply in direction of hair growth to lay any hair around insertion site flat. **Allow to dry completely (10–15 seconds)** until it feels smooth to the touch. Skin protectant helps to further enhance the adherence and holding power of the anchor pad to your skin.

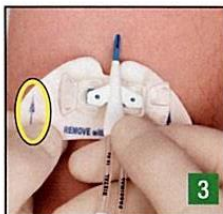


#### Press

3. Align anchor pad so blue directional arrows point toward catheter insertion site.
4. Sliding Posts (if the posts on the StatLock® device retainer slide back and forth) Place suture hole in catheter wing over first post, then slide catheter to capture second post.

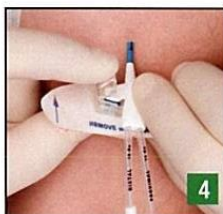

- Fixed Posts (if the posts on the StatLock® device retainer do not slide back and forth or say do not move from side to side)  
Place suture holes in catheter wings over fixed posts.

5. Using thumbs support undersurface of anchor pad and catheter while closing retainer doors. This prevents you from pushing down on your skin.

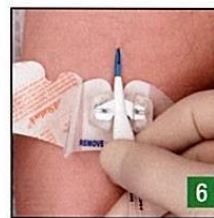

#### Peel & Place

6. Peel away paper backing from anchor pad, one side at a time, (like applying a bandaid), then place on skin.
7. Apply transparent dressing over catheter insertion site and/or StatLock® device anchor pad as instructed by your health care provider.
8. Apply both dressing change labels provided.



#### Catheter Secured

The StatLock® PICC Plus stabilization device should be monitored daily and changed if the pad is lifting, at least every 7 days.